



Prusa Protective Face Shield - RC3



Prusa Research

[VIEW IN BROWSER](#)

updated 25. 3. 2020 | published 20. 3. 2020

Summary

A prototype face shield that we developed. In three days, we went through dozens of prototypes and two verifications with the Czech Ministry of Health.


Version RC3 changelog Changes from RC2:

- Removed hexagonal holes for faster printing speeds
 - With faster printing we use on our farm, some pieces had sharp points and based on testing, the venting is not needed as much
- Several optimizations for stack printing
- New print files with 4 headbands stacked on top of each other
 - Stacking headbands is a great idea by the community, especially useful for overnight printing

Version RC2 Changes from RC1:

- Increased wall thickness (slightly stiffer and more durable)
 - Inner wall from 1.5 mm to 2 mm
 - Outer wall from 2 mm to 2.5 mm
- The headband is no longer printed compressed - Lower pressure on the temple
- The visor was moved further away from the forehead - This allows a better fit over bigger respirators and protective goggles
- The printed part now has an extension for the visor (the visor curves less than the headband).

For now, both RC1 and RC2 versions are relevant. RC1 lets you fit more printed parts on a single print bed. However, if maximum yield isn't the priority, RC2 provides slightly better protection and is more comfortable to wear.

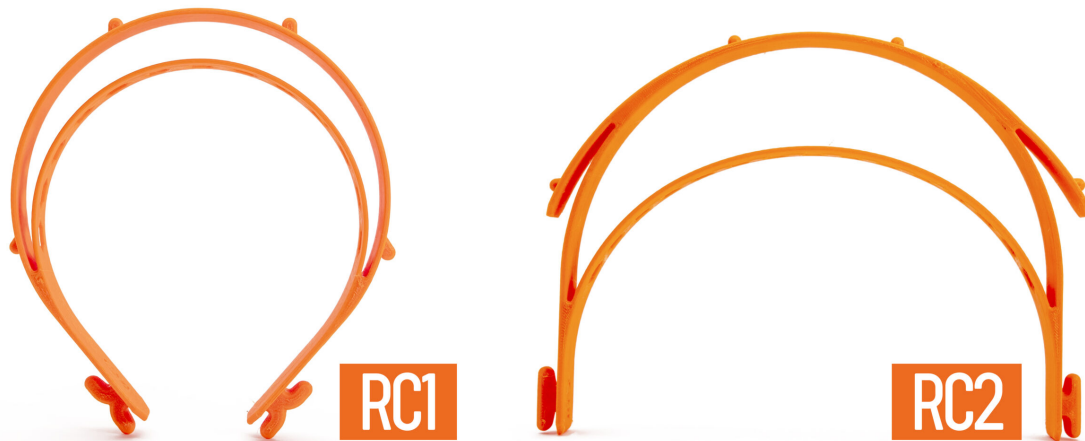

If you want to manufacture shields for others PLEASE READ THIS CAREFULLY

- **Act as if you were infected by the COVID-19 virus.** Wear a face mask and a fresh pair of gloves when collecting each batch of printed parts. Store the parts immediately in a sealable bag.
- Talk with whoever you're making the shields for, let them know about your manufacturing environment
- There is still debate about how long the virus survives on plastic, but most sources mention 2-3 days. That means that by letting the packed face shields sit for 2-3 days before distributing them, you'll greatly reduce risk of transmission
- **Do not store the entire stock in one place**, minimize the risk of cross contamination

More information: <https://blog.prusaprinters.org/from-design-to-mass-3d-printing-of-medical-shields-in-three-days/>

How to STERILIZE the face shields [Sterilization guide](#) We have tests with live COVID running to verify as many of these sterilization methods as possible.

Assembly manual: [English version](#) [English Community version](#) using a **laminating foil** [Czech version](#)

Video:

["https://www.youtube.com/embed/pP7z3iw76GA"](https://www.youtube.com/embed/pP7z3iw76GA)








Rubber band You can either use a thicker one and make a hole in it at each end (make at least 10 mm cut) or use a thin one and tie it to both ends of the shield.

Visor / front plate

- there is a DXF drawing in the file section (and an alternative longer version)
- the holes can be made with a standard office hole puncher
- we used 0.5 mm thick petg sheet (Covestro VIVAK), but you should be able to use any clear laser cuttable plastic with similar thickness.

Visor PDF drawings [Normal size](#) [Extended version \(dentists\)](#)

Licence and selling of face shields We share these files under non-commercial licence. It would be great if you donated these shields to those in need for free. If you need to cover your production costs, we are ok with you selling the shields for production cost. However, we do not want to see these shields on eBay for \$50.

| | | | | | | |
|--|--|--|--|--|--|--|
|  |  |  |  |  |  |  |
| 5.91 hrs | 1 pcs | 0.25 mm | 0.40 mm | 8 | 105.00 g | MK3/S |

[Healthcare](#) > [Accessibility](#)

Supports are not necessary. Print with at least 3 perimeters, about 30% infill.

Ideally, print it from PETG.










Print Files (.gcode)








[↓ DOWNLOAD ALL FILES](#)

| | | |
|---|--|---|
|  | rc2_covid19__2pcs_025mm_petg_mk3s_5 10.9 MB |  |
| | updated 22. 3. 2020 | |
|  |  5.91 hrs |  0.25 mm |
| |  0.40 mm |  PET |
| |  105.20 g |  MK3/S |

Model Files (.stl, .3mf, .obj, .amf)

[↓ DOWNLOAD ALL FILES](#)

| | | | |
|-------|---|----------|---|
| 72x72 | covid19_headband_rc3_1pcs.3mf updated 25. 3. 2020 | 354.1 KB |  |
| 72x72 | covid19_headband_rc2.stl updated 25. 3. 2020 | 662.6 KB |  |
| 72x72 | covid19_headband_rc2_2pcs.3mf updated 25. 3. 2020 | 287.6 KB |  |
| 72x72 | covid19-shield.3mf updated 22. 3. 2020 | 284.2 KB |  |
| 72x72 | covid19_headband_rc2_1pcs.3mf updated 25. 3. 2020 | 290.5 KB |  |
| 72x72 | covid19_headband_rc1.stl updated 22. 3. 2020 | 585.1 KB |  |
| 72x72 | shield_final.dxf updated 25. 3. 2020 | 3.8 KB |  |
| 72x72 | covid19_shield_rc2.stp updated 24. 3. 2020 | 975.6 KB |  |
| 72x72 | bottom_reinforcement.stl updated 25. 3. 2020 | 383.1 KB |  |

| | | | |
|-------|--|----------|---|
| 72x72 | bottom_reinforcement.step updated 24. 3. 2020 | 321.6 KB |  |
| 72x72 | covid19_shiled_r1.step updated 25. 3. 2020 | 841.8 KB |  |
| 72x72 | covid19_headband_quadro_rc3.step updated 25. 3. 2020 | 1.9 MB |  |
| 72x72 | covid19_headband_quadro_rc3.stl updated 25. 3. 2020 | 3.8 MB |  |
| 72x72 | covid19_headband_rc3.stl updated 25. 3. 2020 | 1.5 MB |  |
| 72x72 | shield_longer_version.dxf updated 25. 3. 2020 | 300.2 KB |  |
| 72x72 | covid19_headband_rc3.step updated 25. 3. 2020 | 873.8 KB |  |

License

This work is licensed under a
[Creative Commons \(4.0 International License\)](https://creativecommons.org/licenses/by-nc/4.0/)



Attribution-NonCommercial

- ✘ | Sharing without ATTRIBUTION
- ✓ | Remix Culture allowed
- ✘ | Commercial Use
- ✘ | Free Cultural Works
- ✘ | Meets Open Definition